HANDHABUNGSHINWEIS
INSTRUCTIONS FOR USE
INSTRUCCIONES DE USO
ISTRUZIONE PER L'USO

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English POWEREDGE Blade

Attention

Please read the information in this leaflet carefully. Improper handling and care, as well as unintended use may cause premature wear or risks to patients and users.

Intended use

The POWEREDGE blades serve cutting of biological tissue. They are used with the bipolar coagulation forceps POWEREDGE connected to the bipolar output of an HF-generator and must only be used with the intended parameters.

The maximum admissible operating voltage of the handle must not be exceeded (see information in the catalogue).

Suitable accessories:

Bissinger POWEREDGE REF 82710xxx

Attention: Instruments for HF surgery must only be used by persons specifically trained or instructed for this.

Application and safety notes

- Product and packaging must be inspected for impeccable condition and damage before use.
- Only use products if the packaging is not damaged.
- It is very important that every surgical instrument must be examined for visible damage and wear, e.g. cracks, breaks or defects at the insulation, before every use. Particularly areas such as blades, tips, closures, locks and latches, as well as all moving parts, insulation and ceramics elements must be inspected carefully.
- Never use any damaged instruments.
- Do not use in patients with pacers or other active implants without collecting the corresponding specialist advice.
- Do not use in the presence of flammable or explosive materials.
- The instrument must not be put down on the patient.
- Only active when the contact areas are in sight. Do not touch any other metal instruments.

Sterility

Attention:

STERILE ONLY IF THE PACKAGING IS UNDAMAGED!
The POWEREDGE blades are delivered packed separately and sterile.

If the sterile packaging has been opened, torn or damaged, the POWEREDGE blades must be considered unsterile. This also applies if the expiration date of the POWEREDGE blades has been exceeded. In both cases, the POWEREDGE blades must not be used. POWEREDGE blades are intended for single use. Contaminated products must not be re-conditioned. POWEREDGE blades that have come into contact with blood, tissue or bodily fluids must be disposed of.

Attention: POWEREDGE blades that have already been in contact with the patient or contaminated must not be reconditioned and must be disposed of/rejected.

Attention: For reasons of safety towards our user, the one-time POWEREDGE blades must not be reconditioned. Specifically cleaning and reconditioning of the one-time POWEREDGE blades with sharp edges and fine tips may cause injury of the users.

Preparation

Preparation is not required, since POWEREDGE blades that have already been in contact with the patient or that have been contaminated must not be used again.

Storage

The instruments must be stored in a dry, clean and dust-free environment at moderate temperatures of 5°C to 40°C.

Handling

All instruments should be treated with the greatest care when storing and transporting them. This specifically applies to blades, fine tips and other sensitive areas. Observe that the sterile packaging is not damaged when transporting the instrument. This will cause failure of the instrument.

Warranty

Günter Bissinger Medizintechnik GmbH delivers only inspected and defect-free products to its customers. All of our products are designed and produced to meet the highest quality standards. Liability for products modified as compared to the original, used for any other purpose or treated or used improperly is excluded.

Explanation of symbols

	Batch designation
	Order number
	ATTENTION!
	Observe operating instructions

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Günter Bissinger Medizintechnik GmbH fornisce esclusivamente prodotti omologati e privi di eventuali difetti.
Marcatura CE e numero identificativo del centro designato DQS Medizinprodukte GmbH di Francoforte in Germania