POWERGRIP

Bipolare Koagulationszange
Bipolar Coagulation Forceps

POWERGRIP

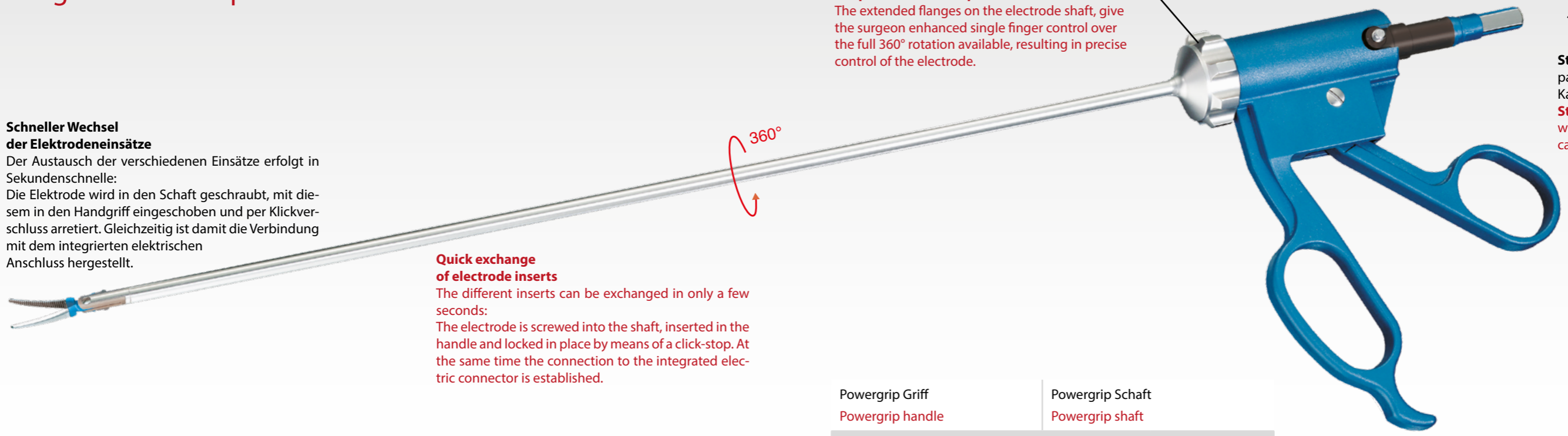
Bipolare Koagulationszange Bipolar Coagulation Forceps

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Quick exchange of electrode inserts
The different inserts can be exchanged in only a few seconds: The electrode is screwed into the shaft, inserted in the handle and locked in place by means of a click-stop. At the same time the connection to the integrated electric connector is established.

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Precise coagulation, more powerful grasping, better cutting and more exact work: the Bissinger POWERGRIP

The Bissinger POWERGRIP is an instrument used for grasping, cutting and bipolar coagulation in Minimally Invasive Surgery. It is characterized by a mechanism enabling well adjustable opening and closing of the jaws with very high pressure while grasping and cutting. This is accompanied by precise coagulation of the tissue. The jaws are moved by a double hinge which is connected electrically and mechanically to the handle by a slide rod. The connector for standardized bipolar cables is integrated in the particularly ergonomic handle made of high quality PEEK polymer. This saves the purchase of expensive special cables and prevents them from being connected incorrectly. We supply standardized bipolar cables for connection to practically all common high-frequency generators. The Bissinger POWERGRIP can be dismantled in three parts. This enables thorough and reliable cleaning and facilitates the execution of repair work or the replacement of defective parts. Beyond that, the Bissinger POWERGRIP offers a number of important further advantages:

- exchangeable electrode inserts for different applications
- easy handling
- excellent cleaning and ease of repair

	Große Fasszange Large grasping forceps	82400010
	Pinzette gebogen Forceps, curved	82400011
	Fasszange gebogen Grasping forceps, curved	82400015
	Delta Fasszange Delta Grasper	82400014

	Dissektor Maryland gebogen Maryland dissector, curved	82400019
	Schere gebogen Scissors, curved	82400018
	Hakenzange Claw forceps	82400012
	Duck Bill Fasszange Duck Bill Grasper	82400008

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Bissinger – Focus on better solutions

Bipolar and monopolar instruments
 Accessories for electrosurgery
 Surgical instruments
 Design and development of instruments
 to customer specifications
 Service and repair

2-Bananenstecker* 	3 m	Art. No. 80100010
	5 m	Art. No. 80100013
U20 	4 m	Art. No. 80100260
Erbe 	3 m	Art. No. 80100017
	5 m	Art. No. 80100018
Martin, Berchtold Aesculap GK 55, GK 60 	3 m	Art. No. 80100019
	5 m	Art. No. 80100020
Valleylab, Lamidey, EMC 	3 m	Art. No. 80100021
	5 m	Art. No. 80100022

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Medizintechnik GmbH